
venome

Biorevitalization

H₂O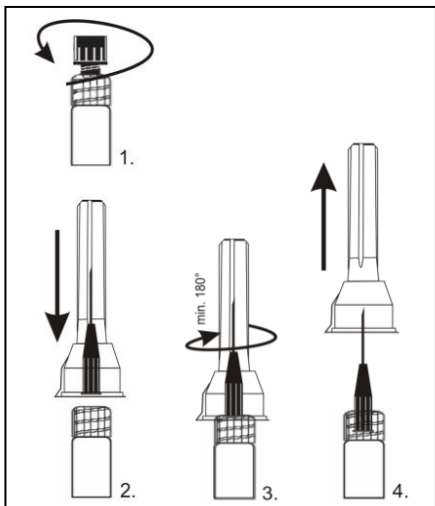

CE 0297

S & V Technologies GmbH
 Neudorfstraße 20a 16761 Hennigsdorf,
 Germany
 Tel.: +49 3302-202-7900
 Fax: +49 3302-202-7905
 e-mail: info@skin-vision.eu

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Biorevitalization H₂O

Medical device according to 93/42/EEC

Composition
1 ml contains 12 mg non-crosslinked hyaluronate.

Content
One sterile, single-use syringe containing Venome Biorevitalization H₂O; two sterile needles 32G1/2 for single-use and intended exclusively for the injection of Venome Biorevitalization H₂O; product leaflet, as well as a set of labels of which one is provided for the patient and the other for the medical record.

Indication
Viscoaugmentation for correction of volume losses or defects related to disease or age in face, neck and décolleté, e.g. for skin rejuvenation (bio-revitalisation) and treatment of fine lines.

Note
Intradermal injection of Venome Biorevitalization H₂O must only be performed by physicians with appropriate training in dermal filler injection. The correct injection technique is essential for the success of the treatment. Injection into the wrong skin layer, an inappropriate technique or a high injection rate may lead to pronounced side effects.

Properties

Venome Biorevitalization H₂O is a monophasic, sterile, isotonic, apyrogenic clear gel of highly purified hyaluronic acid of biofermentative origin and thus free of animal traces. Hyaluronic acid, a natural polysaccharide, is an important structural element in skin, in subcutaneous and connective tissue, in synovial tissue as well as in synovial fluid. It belongs to the few substances that are chemically identical in all living organisms.

Mode of action

With age, the concentration of hyaluronic acid in the skin decreases significantly. As a consequence, the skin loses elasticity and tonus, and because of the decrease of skin turgor, wrinkles start to develop. Addition of Venome Biorevitalization H₂O to the skin tissue results in restoration of the extracellular matrix and in rehydration, due to the high water-retention capacity of hyaluronic acid. Elasticity and turgor of the skin are restored, fine wrinkles are smoothed and the skin appears younger. In addition, it is assumed that hyaluronic acid is capable of stimulating collagen synthesis and trapping radicals, thereby retarding the skin-aging process. The degradation of hyaluronic acid occurs enzymatically through the endogenous enzyme hyaluronidase as well as by oxidative hydroxylation.

Application and Techniques

Venome Biorevitalization H₂O should be used at room temperature. It has been developed for mesotherapy. This gel is used for hydration and rejuvenation of the dermis, but is not suitable for the filling of wrinkles. It is applied by extensive injections into the upper dermis of face, neck, décolleté and hands. The use of a mesotherapy injection gun allows a time-saving, uniform distribution. The duration is approx. 3 months. In the course of mesotherapy, this treatment should be performed 3 x at intervals of approx. 2-3 weeks to achieve the desired result of skin rejuvenation. For maintenance, touch-up injections twice a year are sufficient.

Contraindications

Venome Biorevitalization H₂O is contraindicated in case of:

- allergy to hyaluronic acid
- auto-immune disease or during immunotherapy
- acute skin infection in the area to be treated (e.g. acne)
- acute influenza and feverish cold
- acute systemic infection (herpes, psoriasis)
- coagulation disorder
- a tendency to form scars
- pregnancy or breast-feeding
- people under age of 18

Venome Biorevitalization H₂O must not be injected into:

- blood vessels
- bone joints
- breasts

Precautions

As a rule, the physician should carefully investigate the patient history with respect to diseases, other anti-aging treatments and individual factors that might increase the risk of adverse reactions. The patient should be informed prior to treatment about the concept of tissue augmentation with hyaluronic acid, common injection-related reactions and possible side effects. Venome Biorevitalization H₂O should only be injected into healthy, non-inflamed and disinfected areas of skin. It is recommended to pretreat the injection area with an anaesthetic cream or patch. Patients should not take aspirin, ginkgo preparations or other blood thinning substances one week prior to treatment. After treatment the patient should not expose to excessive sun or intense heat (solarium, sun bath, sauna), at least until the initial reactions (redness, swelling) have resolved.

Side effects

Immediately after treatment, injection related reactions like redness and swelling and in rare cases itching or bruising may be observed at the injection sites. Allergic reactions are extremely rare because Venome Biorevitalization H₂O is protein free due to the non-animal source. Injection-related reactions will dissolve spontaneously within a short while. In case of an inflammatory reaction it is recommended that the patient visit the physician.

In literature, individual cases of granuloma, abscesses, discoloration and necrosis have been described.

Warnings

Venome Biorevitalization H₂O is only suitable for intradermal injection. Due to the lack of clinical comparability data, it should not be mixed with other manufacturer's products (e.g. tissue fillers). It should not be combined with other methods of treatment of aesthetic medicine such as peeling, dermabrasion or any form of treatment with waves (laser, radiofrequency, ultrasound). Venome Biorevitalization H₂O should not be used together with products that contain quaternary ammonium compounds because of possible incompatibility. Before use, the integrity of the packaging and the expiry date of the product have to be checked. If the 32G1/2 needle gets blocked, the injection should be interrupted and the needle should be replaced. Do not use if packaging is damaged. For single use only. Discard any unused amounts. Re-use may pose a risk of infection caused by contaminated sterile product. Do not resterilize. The packaging is not autoclavable. Do not use in the operating room.

Storage

Between 2°C and 25°C, in a dry place, protected from sun light.

Disposal

Directly after use dispose of syringe with unused gel and needles.

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